



ATTENTION



- The Hybrid Power Rod Cutter, PI-2477, is designed to **ONLY** cut Titanium or Cobalt Chromium rods of circular diameters 5.5mm, 6.0mm, and 6.35mm
- Do **NOT** cut other shapes, sizes, or materials
- Ensure rods are **ONLY** cut in the hole of matching diameter size
- Operating this cutter requires a moderate-high amount of strength; ensure user is physically able to exert enough force for proper operation
- Misuse may result in failure and is easily detectable

P·A·C·I·F·I·C®
INSTRUMENTS